FRANÇAIS

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Air Liquide Medical Systems S.r.l.
Département technique

MASQUE	DÉCONTAMINATION	DÉSINFECTION THERMIQUE	DÉSINFECTION CHIMIQUE	STÉRILISATION	CYCLES HOMOLOGUÉS
Respireo N	✓	N.A.	N.A.	N.A.	N.A.
Respireo Primo N	✓	N.A.	N.A.	N.A.	N.A.
Respireo Primo N Non Vented	✓	✓	✓	✓	20
Respireo Primo F	✓	N.A.	N.A.	N.A.	N.A.
Respireo Primo F Non Vented	✓	N.A.	N.A.	N.A.	N.A.
Respireo Hospital F Non Vented reusable*	✓	✓	✓	✓	20
Respireo Blue Non Vented elbow	✓	✓	✓	✓	20
Respireo SOFT Baby**	✓	✓	✓	✓	12
Respireo SOFT Baby Non Vented**	✓	✓	✓	✓	12
Respireo SOFT Child**	✓	✓	✓	✓	12
Respireo SOFT Child Non Vented**	✓	✓	✓	✓	12
Respireo SOFT Nasal	✓	✓	✓	N.A.	20
Respireo SOFT Nasal Silent	✓	✓	✓	N.A.	20
Nasal Non Vented elbow	✓	✓	✓	✓	20
ALNEST / INEVA N1 Silent	✓	✓	✓	N.A.	20
ALNEST / INEVA N1	✓	✓	✓	N.A.	20
Respireo SOFT Facial	✓	✓	✓	N.A.	20
Respireo SOFT Facial Non Vented	✓	✓	✓	✓	20
ALNEST / INEVA F1	✓	✓	✓	N.A.	20
ALNEST / INEVA F1 Non Vented	✓	✓	✓	✓	20

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ENGLISH

Decontamination, disinfection and sterilization guide for masks.

This guide defines the procedures validated by Air Liquide Medical Systems S.r.l. to be performed only in hospital or clinical structures for the regeneration of the masks in order to make them suitable to be reused in compliance with UNI EN ISO 17664:2021. It is up to the person in charge of the regeneration processes to ensure that the operations are performed using suitable materials, devices and personnel in order to achieve the desired result. This generally requires validation and systematic monitoring of the entire process. The manufacturer does not respond in case the process does not comply with the instructions provided, both for its actual effectiveness or any possible undesirable consequences.

Air Liquide Medical Systems S.r.l.
Technical department

MASK	DECONTAMINATION	THERMAL DISINFECTION	CHEMICAL DISINFECTION	STERILISATION	VALIDATED CYCLES
Respireo N	✓	N.A.	N.A.	N.A.	N.A.
Respireo Primo N	✓	N.A.	N.A.	N.A.	N.A.
Respireo Primo N Non Vented	✓	✓	✓	✓	20
Respireo Primo F	✓	N.A.	N.A.	N.A.	N.A.
Respireo Primo F Non Vented	✓	N.A.	N.A.	N.A.	N.A.
Respireo Hospital F Non Vented reusable*	✓	✓	✓	✓	20
Respireo Blue Non Vented elbow	✓	✓	✓	✓	20
Respireo SOFT Baby**	✓	✓	✓	✓	12
Respireo SOFT Baby Non Vented**	✓	✓	✓	✓	12
Respireo SOFT Child**	✓	✓	✓	✓	12
Respireo SOFT Child Non Vented**	✓	✓	✓	✓	12
Respireo SOFT Nasal	✓	✓	✓	N.A.	20
Respireo SOFT Nasal Silent	✓	✓	✓	N.A.	20
Nasal Non Vented elbow	✓	✓	✓	✓	20
ALNEST / INEVA N1 Silent	✓	✓	✓	N.A.	20
ALNEST / INEVA N1	✓	✓	✓	N.A.	20
Respireo SOFT Facial	✓	✓	✓	N.A.	20
Respireo SOFT Facial Non Vented	✓	✓	✓	✓	20
ALNEST / INEVA F1	✓	✓	✓	N.A.	20
ALNEST / INEVA F1 Non Vented	✓	✓	✓	✓	20

* For Respireo Hospital F Non Vented Reusable, in order to perform a complete cleaning of the regulation screw is necessary to completely rotate counterclockwise the knob.

** For Respireo SOFT Baby and Respireo SOFT Child, in both Vented and Non Vented versions, the tube assembly cannot be disinfected by high-level thermal disinfection or sterilised.

PHASE 1	PREPARATION	Disassemble entirely the mask according to the instructions for use of the specific device. The headgear is not suitable for regeneration, so it shall be replaced.
PHASE 2	DECONTAMINATION	<p>Perform DECONTAMINATION always and with care before choosing between THERMAL DISINFECTION or CHEMICAL DISINFECTION.</p> <ul style="list-style-type: none"> Soak the components for at least 15 minutes in water containing a neutral pH detergent (i.e. gigazyme[®] with 1% dilution) then clean each of them for one minute with a soft-bristle brush. Rinse all the components thoroughly in running tap water. Air-dry the components away from direct sunlight.
PHASE 3A	HIGH LEVEL THERMAL DISINFECTION	<ul style="list-style-type: none"> Using a certified automatic hot water disinfection system, soak the components in a solution of a neutral pH detergent, in combination with a neutral pH rinsing agent if deemed appropriate. In compliance with UNI EN ISO 15883:2014, the cleaning cycle must comply with one of the following temperature-time combinations: 70°C - 100 minutes 80°C - 10 minutes 90°C - 1 minute Air-dry the components away from direct sunlight.
PHASE 3B	HIGH LEVEL CHEMICAL DISINFECTION	<ul style="list-style-type: none"> Soak the components in one of the following high-level disinfectant solutions, following the instructions provided by the solution producer: - CIDEX[®] OPA Rinse the components thoroughly in tap water for at least one minute. Perform this operation 3 times, using clean water each time. Air-dry the components away from direct sunlight.
PHASE 4	AUTOCLAVE STERILIZATION	<ul style="list-style-type: none"> Sterilise the components in an autoclave at 121°C (250°F) for no longer than 30 minutes. Drying takes place during the sterilization process.
PHASE 5	INSPECTION	<p>Visually inspect each part of the mask.</p> <p>Note: If any component shows signs of deterioration (e.g. cracks, scratches, fractures or tears), it should be discarded and replaced. Slight discolouration of the silicone components is quite normal.</p>
PHASE 6	REASSEMBLING	Reassemble the mask according to the instructions for use of the specific device.
PHASE 7	CONSERVATION	Store in a dry and dust free environment at temperature between -25°C and +50°C, away from direct sunlight.

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Air Liquide Medical Systems S.r.l.
Tekninen osasto

MASKI	DEKONTAMINAATIO	LÄMPÖDESINFIOINTI	KEMIALLINEN DESINFIOINTI	STERILOINTI	HYVÄKSYTYT SYKLIT
Respireo N	✓	Ei saatavilla	Ei saatavilla	Ei saatavilla	Ei saatavilla
Respireo Primo N	✓	Ei saatavilla	Ei saatavilla	Ei saatavilla	Ei saatavilla
Respireo Primo N Non Vented	✓	✓	✓	✓	20
Respireo Primo F	✓	Ei saatavilla	Ei saatavilla	Ei saatavilla	Ei saatavilla
Respireo Primo F Non Vented	✓	Ei saatavilla	Ei saatavilla	Ei saatavilla	Ei saatavilla
Respireo Hospital F Non Vented reusable*	✓	✓	✓	✓	20
Respireo Blue Non Vented elbow	✓	✓	✓	✓	20
Respireo SOFT Baby**	✓	✓	✓	✓	12
Respireo SOFT Baby Non Vented**	✓	✓	✓	✓	12
Respireo SOFT Child**	✓	✓	✓	✓	12
Respireo SOFT Child Non Vented**	✓	✓	✓	✓	12
Respireo SOFT Nasal	✓	✓	✓	Ei saatavilla	20
Respireo SOFT Nasal Silent	✓	✓	✓	Ei saatavilla	20
Nasal Non Vented elbow	✓	✓	✓	✓	20
ALNEST / INEVA N1 Silent	✓	✓	✓	Ei saatavilla	20
ALNEST / INEVA N1	✓	✓	✓	Ei saatavilla	20
Respireo SOFT Facial	✓	✓	✓	Ei saatavilla	20
Respireo SOFT Facial Non Vented	✓	✓	✓	✓	20
ALNEST / INEVA F1	✓	✓	✓	Ei saatavilla	20
ALNEST / INEVA F1 Non Vented	✓	✓	✓	✓	20

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MASKA	ODKAŻANIE	DEZYNFEKCJA TERMICZNA	DEZYNFEKCJA CHEMICZNA	STERYLIZACJA	CYKLE HOMOLOGOWANE
Respireo N	✓	N.D.	N.D.	N.D.	N.D.
Respireo Primo N	✓	N.D.	N.D.	N.D.	N.D.
Respireo Primo N Non Vented	✓	✓	✓	✓	20
Respireo Primo F	✓	N.D.	N.D.	N.D.	N.D.
Respireo Primo F Non Vented	✓	N.D.	N.D.	N.D.	N.D.
Respireo Hospital F Non Vented reusable*	✓	✓	✓	✓	20
Respireo Blue Non Vented elbow	✓	✓	✓	✓	20
Respireo SOFT Baby**	✓	✓	✓	✓	12
Respireo SOFT Baby Non Vented**	✓	✓	✓	✓	12
Respireo SOFT Child**	✓	✓	✓	✓	12
Respireo SOFT Child Non Vented**	✓	✓	✓	✓	12
Respireo SOFT Nasal	✓	✓	✓	N.D.	20
Respireo SOFT Nasal Silent	✓	✓	✓	N.D.	20
Nasal Non Vented elbow	✓	✓	✓	✓	20
ALNEST / INEVA N1 Silent	✓	✓	✓	N.D.	20
ALNEST / INEVA N1	✓	✓	✓	N.D.	20
Respireo SOFT Facial	✓	✓	✓	N.D.	20
Respireo SOFT Facial Non Vented	✓	✓	✓	✓	20
ALNEST / INEVA F1	✓	✓	✓	N.D.	20
ALNEST / INEVA F1 Non Vented	✓	✓	✓	✓	20

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