

Respireo **SOFT**
Nasal

Respireo **SOFT** Nasal
Silent



Enjoy your sleep better

The key to a successful treatment of your patients.

The Respireo™ SOFT Nasal range will make life easier for your patients by providing an optimal user experience.

Univeo™ headgear

- Ensures a good peripheral stability and an homogeneous distribution of support points

Respireo™ SOFT cushion

- Naturally fits the nasal bridge to ensure a seal and comfort throughout sleep

Ergonomic hooks

- Make it easier for patients to put on and take off the headgear quickly

Flexible lateral arms

- To adjust to the width of patients' faces

A thin and flexible frame


- Ensures an optimal contact of the cushion on the patient's face

360° rotating elbow

- Facilitates the patient's movements

Also available with the Peace System® silent leak

- This ventilation port ensures a smoother airflow and a more efficient diffusion

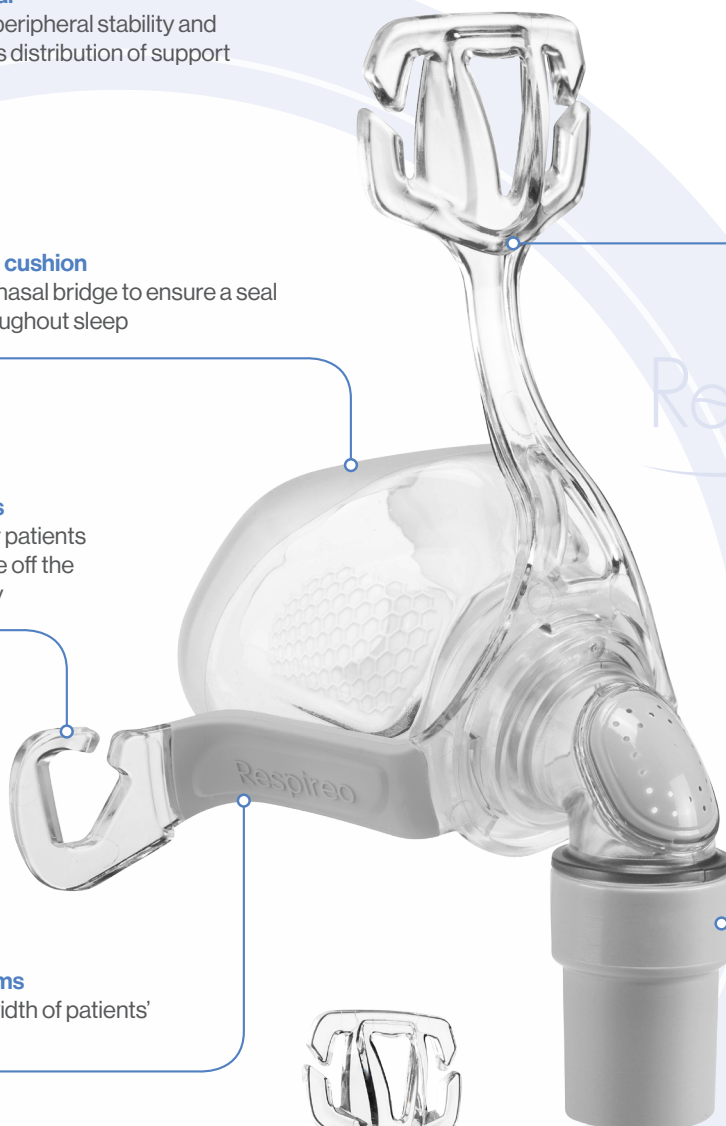

Respireo SOFT Nasal
Silent

Respireo SOFT
Nasal

Respireo **SOFT** Nasal

Respireo **SOFT** Nasal Silent

Its main strengths

Comfortable

- Cushion with SOFT technology fitting the contours of the nose to reduce pressure on the nasal bridge, which is a highly sensitive area


Easy to use

- Easy to install, adjust and clean
- Conservation of the headgear settings predefined by the healthcare provider thanks to the hooks

- Provides a peaceful sleep for the patient and his/her partner thanks to the optimal diffusion of the expiratory flow


Silent diffusion

Fitting guide

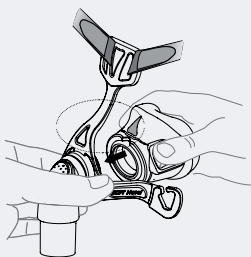


1

Using the mistake-proofing, assemble the mask with the cushion.

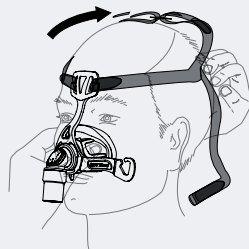


2

Position the mask on the nose while pulling the headgear over the head.

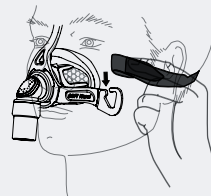

3

Attach the lower straps of the headgear into the hooks.

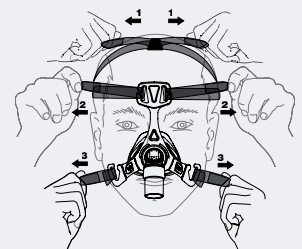

4

Adjust the settings of the headgear from top to down (1,2 then 3).

Tips: Ensure not to tight too much the headgear.

References

Respireo™ SOFT Nasal

Product Reference	Description	Size	Distributor code
472360	Respireo™ SOFT Nasal	M	KM227500
472361	Respireo™ SOFT Nasal	L	KM227600

Spare parts:

472471	UNIVEO™ headgear	One size	KM227700
472470	Respireo™ SOFT Nasal cushion	M	KM227800
472473	Respireo™ SOFT Nasal cushion	L	KM228700
472474	Nasal Non Vented elbow*	-	KM228800
472483	Grey swivel fitting**	-	KM229130

Respireo™ SOFT Nasal Silent

Product Reference	Description	Size	Distributor code
472365	Respireo™ SOFT Nasal Silent	M	KM229106
472366	Respireo™ SOFT Nasal Silent	L	KM229107

Spare parts:

472471	UNIVEO™ headgear	One size	KM227700
472470	Respireo™ SOFT Nasal cushion	M	KM227800
472473	Respireo™ SOFT Nasal cushion	L	KM228700
472474	Nasal Non Vented elbow*	-	KM228800
472483	Grey swivel fitting**	-	KM229130

Scan here for
more information



Manufactured by

AIR LIQUIDE MEDICAL SYSTEMS S.r.l
Via Bisceglie, 66 - 20152 Milano (MI) - ITALY
PLANT: Via dei Prati, 62 - 25073 Bovezzo (BS) - ITALY
Tel. +39 030.201.59.11
Fax +39 030.209.83.29

Distributor Contact

AIR LIQUIDE MEDICAL SYSTEMS S.A.
Parc de Haute Technologie
6, rue Georges Besse
92182 Antony Cedex
France

medicaldevice.airliquide.com

Hotline ALMS: +33 179 51 70 01

Respireo™ SOFT Nasal is a reusable nasal mask with calibrated exhalation orifices (Vented), intended to be used at home or in a hospital setting by multiple adult patients (weighing more than 30 kg) who have been prescribed for a non-invasive positive pressure ventilation therapy (NPPV), for example a continuous positive airway pressure therapy (CPAP) or Bilevel positive airway pressure therapy (BiPAP).

Respireo™ SOFT Nasal Silent is a reusable nasal mask with calibrated exhalation orifices (Vented), intended to be used at home or in a hospital setting by multiple adult patients (weighing more than 30 kg) who have been prescribed for a non-invasive positive pressure ventilation therapy (NPPV), for example a continuous positive airway pressure therapy (CPAP) or Bilevel positive airway pressure therapy (BiPAP).

Medical devices class IIa CE0051. Please read the user manual.

* Medical device that changes the intended use of the mask into the following one:
Respireo™ SOFT Nasal and Respireo™ SOFT Nasal Silent with Nasal Non Vented Elbow connector mounted are reusable nasal masks without calibrated exhalation holes intended to be used at home or in a hospital setting by multiple adult patients (weighing more than 30 kg) under Non invasive Positive-Pressure Ventilation (NPPV).

** This spare part is only for the Vented version.